

**DIRECTIONS**

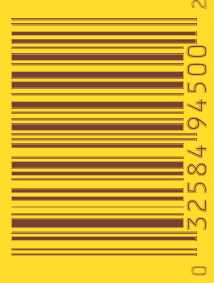
1. Clean end of teat with soap and water or mild antiseptic. Insert dilator in teat and allow it to remain until next milking, then remove dilator and discard. Discard first few streams of milk in strip cup, then milk out quarter.




2. Clean and disinfect teat each time before inserting dilator. Use a new dilator in teat between milkings until teat milks free by hand.  
**CAUTION:** Keep out of reach of children.  
Clean udder and teats thoroughly before each milking. Do not remove cylinder from this box.

4. Désinfecter le trayon chaque fois avant d'introduire le dilateur, et utiliser un nouveau dilateur entre les traite. Jusqu'à ce que le trayon se traite bien à la main.  
**AVIS:** Nettoyer le pis et les trayons à fond avant chaque traite. Garder hors d'atteinte des enfants.  
Ne pas retirer de cette boîte le cylindre.

**MODE D'EMPLOI**

1. Nettoyer l'extrémité du trayon au savon et à l'eau ou un antiseptique doux.  
2. Introduire le dilateur dans le trayon et l'y laisser jusqu'à la prochaine traite.  
3. A la prochaine traite, l'enlever et le jeter. Jeter les quelques premiers jets de lait dans un gobelet d'égoûtage, puis traiter le quartier à fond.



**MY-764 - Bottom Tray**  
Size: 2-3/8 x 1-3/8 x 2-3/8 w/ 2" Hole  
Prints:  
208N80103 Brown   
208N80104 Buff   
and Spot Aqueous   
(void for glue only)

**The Mid-York Press, Inc.**  
**Note:** PMS spot colors on color proof may or may not be accurate to Pantone Matching System.